Silhouette HA Body

A highly purified, monophasic body filler based on naturally cross-linked hyaluronic acid, designed for safe and effective body contouring



Maximum effect with minimal intervention:

restore firmness, volume, and sculpted contours with natural-looking results

Contents

- What is Meamo labs Silhouette HA Body?
- Who Can Benefit from Meamo Labs Silhouette HA Body?
- 11 What Makes Meamo labs Silhouette HA Body Unique?
- Where and How to Use Meamo labs Silhouette HA Body?
- What to Do After Meamo labs Silhouette HA Body Injection Sessions?
- How to Store Meamo labs Silhouette HA Body?

What is Meamo labs Silhouette HA Body?



Meamo Labs Silhouette HA Body is a nextgeneration injectable filler designed to restore lost soft tissue volume and reshape body contours. It is administered into the deep dermis and subcutaneous fat layer, filling volume deficiencies and creating smooth, natural-looking shapes.



Volume loss can result from aging, weight loss, hormonal changes, or post-pregnancy body transformations. Meamo Labs Silhouette HA Body effectively restores this lost volume, bringing back density, firmness, and a balanced, aesthetically pleasing silhouette.

Its formula is based on stabilized hyaluronic acid, combined with ultrashort peptides, glaucine, and fermented collagen, providing not only precise contouring but also long-lasting biological support for tissue regeneration.

The product is biocompatible, safe, and delivers prolonged results without a foreign-body effect.

Define Your Body's Natural Contours

Who Can Benefit from Meamo Labs Silhouette HA Body?

Who Can Benefit from Silhouette HA Body?



SAGGING BREASTS

With age, pregnancy, or breastfeeding, breast tissue can lose firmness and shape. Meamo Labs Silhouette HA Body helps restore definition and lift the upper pole of the breast without surgery



LOSS OF TISSUE VOLUME

Rapid weight loss, aging, or individual body structure may lead to volume depletion in the hips, buttocks, breasts, or calves. The filler restores lost volume, enhancing body curves in a natural and balanced way



LOOSE, SAGGING SKIN

Reduced skin elasticity leads to flaccidity and drooping. Silhouette HA Body stimulates collagen production and strengthens the dermal matrix, tightening and firming the skin



LOSS OF FIRMNESS AFTERPREGNANCY OR DIETING

Hormonal shifts, skin stretching, and rapid weight loss often distort body contours. The filler helps restore tone and elasticity, improving skin texture and body shape

Who Can Benefit from Silhouette HA Body?



POORLY DEFINEDOR FLAT BODY CONTOURS

Some individuals naturally lack prominent curves, while others lose definition with age. Meamo Labs Silhouette HA Body sculpts clear, aesthetically pleasing lines for a more harmonious silhouette



ASYMMETRY ORUNDERDEVELOPED AREAS

Uneven or disproportionate body zones —such as breasts, buttocks, or calves — can be corrected non-surgically. The filler allows precise contouring to balance asymmetry and enhance overall proportions without implants

Expert-Recommended for Body Volume Restoration.


Aesthetic doctors recommend **Meamo Labs Silhouette HA Body** to individuals concerned about volume loss or skin laxity in various areas of the body — including the arms, thighs, buttocks, breasts, and calves.

The filler is especially effective for **reshaping the breasts** and buttocks with natural-looking results.


With Meamo Labs Silhouette HA Body, you can:




Avoid plastic surgery and implants for breast or buttock augmentation

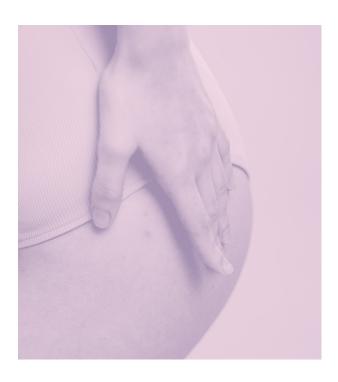

Instantly enhance breast or buttock volume and quickly return to your daily routine

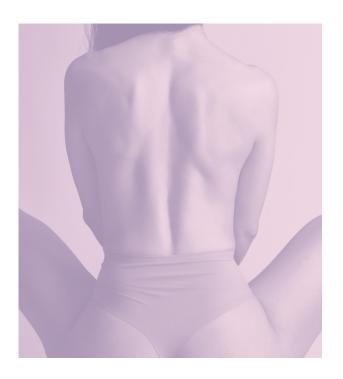

Achieve natural, sculpted contours through a minimally invasive injectable procedure

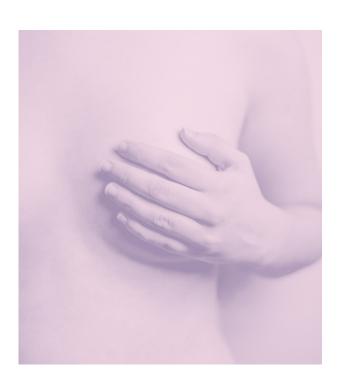

Restore breast shape and upper volume after pregnancy or breastfeeding

With Meamo Labs Silhouette HA Body, you can:



Bring back volume and firmness lost due to aging, weight loss, or body transformation



Correct asymmetry — balance uneven volume or shape in the breasts, buttocks, thighs, or calves



Improve overall body silhouette and definition — without an overfilled or artificial look



Firm and tone areas with loose or sagging skin for a more youthful appearance

What Makes Silhouette HA Body Unique?

The Most Natural Form of Hyaluronic Acid!
Thanks to a **patented stabilization technology**, the hyaluronic acid in **Meamo Labs Silhouette HA Body** retains its **natural 3D cross-linked structure** with **minimal synthetic cross-linking agents**. This innovative approach ensures:

High biocompatibility — smooth and safe integration into tissue

Natural-looking results — volume appears balanced, with no foreign-body effect

Minimal risk of rejection or inflammation

Additionally, the formula combines both long- and short-chain hyaluronic acid molecules, which allows for:

- Even distribution within tissues no lumps or irregularities
- Smooth, natural texture mimics the feel of real tissue
- Slow and consistent degradation, providing a long-lasting volumizing effect



Precision in Shape. Ideal Flexibility Why Monophasic Structure Matters?

Meamo Labs Silhouette HA Body is a monophasic, homogeneous, and highly elastic filler — and this sets it apart from traditional biphasic alternatives not only in composition, but in the quality of results:

Smooth and controlled injection allows for precise shaping and contouring

Natural results even with minimal volume

3.

Optimal viscoelasticity maintains form in high-mobility areas like the buttocks, thighs, and breasts

Superior flexibility and stability the filler resists shifting or deformation under pressure

Maximum Safety. Minimal Chemicals. Maximum Trust. Meamo Labs Silhouette HA Body contains less than 0.2 ppm of BDDE, making it one of the lowest BDDE concentrations among body fillers. This ensures:

- Low risk of side effects
- Non-toxic formulation
- Minimal chance of allergic reactions or fibrosis






^{*} The product is suitable even for sensitive patients and is safe for use in high-volume treatment areas

The Four Pillars of Your Beauty What Makes Meamo labs Silhouette HA Body So Effective?

Meamo Labs Silhouette HA Body combines four powerful active components — each one enhancing volume, firmness, and tissue regeneration. Their synergy delivers not just a visible transformation, but also long-term biological support for skin health and structure.

Hyaluronic Acid (HA): The Foundation of Volume and Hydration

- Instantly restores lost volume by attracting and retaining moisture
- Stimulates fibroblast activity boosting collagen and elastin production
- Improves skin elasticity and reinforces the dermal matrix
- Enhances microcirculation and oxygen delivery to skin cells
- Provides antioxidant protection against oxidative stress

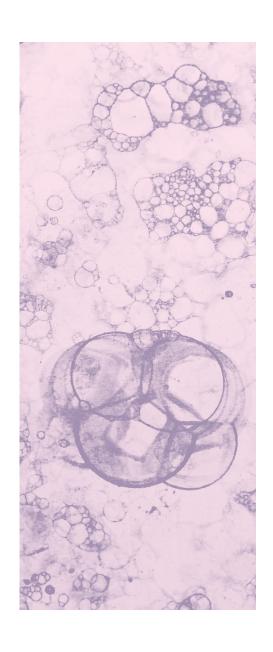

^{*} Thanks to patented stabilization technology, the hyaluronic acid in this filler retains its **natural**, **interwoven 3D structure**, allowing for smooth tissue integration and a long-lasting effect

Ultrashort Peptides — Activators of Renewal and Lifting


- Epigenetically regulate gene expression to stimulate the synthesis of collagen, elastin, and fibronectin
- Enhance dermal lifting and increase the skin's resistance to stretching
- Improve the skin's moisture retention, helping to prolong the filler's volumizing effect
- Provide strong antioxidant and anti-inflammatory properties, reducing post-procedure swelling and redness
 - Promote tissue regeneration and angiogenesis, especially important in
- low-vascularized areas such as the buttocks or calves



The formula includes **tripeptides**, **tetrapeptides**, **hexapeptides**, **oligopeptides**, and **polypeptides**, including **EGF** (epidermal growth factor) for optimal skin repair and renewal.


Fermented Collagen Filtrate with Lactobacillus — Regeneration Stimulator

- Contains polycollagen peptides that are easily absorbed by skin cells
- Stimulates the natural regeneration of connective tissue, enhancing skin integrity
- Increases skin density and elasticity, smoothing out the surface for a more youthful appearance

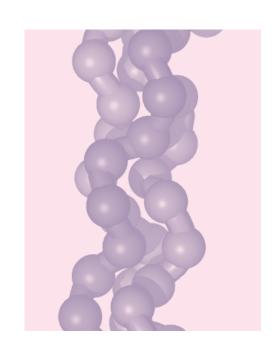

Glaucine — Comfort and Regeneration

- A natural alkaloid with mild analgesic and muscle-relaxing effects
- Reduces inflammation and tissue sensitivity, increasing procedure comfort
- Promotes more even distribution of the filler and better integration into the tissues

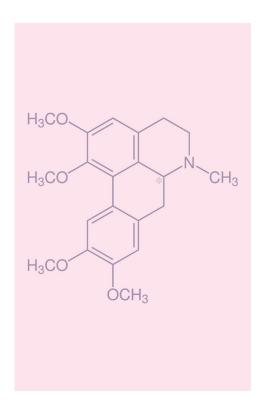

Where and How to Use Meamo labs Silhouette HA Body?



Areas of Application of Meamo labs Silhouette HA Body:



BUTTOCKS —
 volume restoration, shape correction, and symmetry improvement



 THIGHS (OUTER AND INNER AREAS) smoothing lines and adding roundness



CALVES —
 restoring lost volume
 and improving symmetry



 BREAST (UPPER POLE) gentle lifting and postbreastfeeding restoration

Areas of Application of Meamo labs Silhouette HA Body:



ARMS (UPPER ARM AREA) —
 correcting sagging skin
 and volume loss, especially
 after weight loss or age-related
 changes



 LOWER BACK AREA (ABOVE THE BUTTOCKS) —
 evening out transitions, emphasizing the "hourglass" silhouette



 DÉCOLLETAGE AREA smoothing wrinkles, improving skin density and elasticity




SUBGLUTEAL FOLD —
 creating a sharper
 and smoother transition
 between the buttocks
 and thighs (popular
 for S-curve modeling)


Important: It is not recommended to inject the filler into the lower parts of the breasts or buttocks, as it can cause ptosis (sagging of tissues).


1.

2.

3.

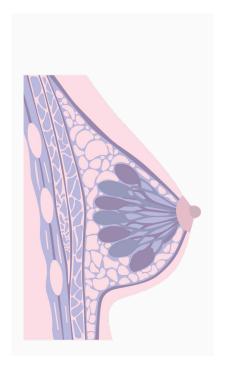

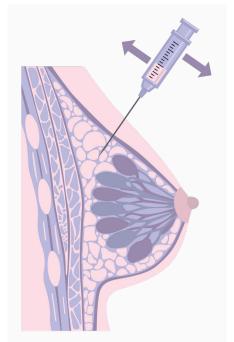

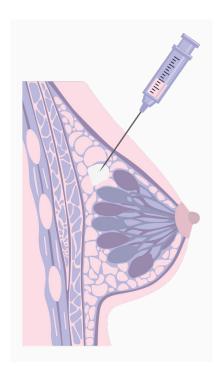

- Injection Technique
- · A blunt cannula of 16G-18G is used
- The cannula is inserted into the fat layer, and a tunnel is created with gentle oscillating and rotating movements
- Inject the filler slowly while withdrawing the cannula, ensuring even distribution of the product
- Injection Depth
- Deep layers of the dermis or subcutaneous fat
- It is important to avoid injecting into the muscle layer, as hyaluronic acid will quickly break down in this case, significantly reducing the effect

Dilution of the Product

Meamo labs Silhouette HA Body can be used in its pure form, especially when significant volume and firm correction are required. However, if difficulties arise during aspiration or injection (e.g., due to high viscosity), it is permissible to dilute the product with a physiological saline solution (0.9% NaCl) prior to use

When and Why to Dilute

Dilution helps reduce the filler's density, improving its flowability and distribution in tissues, particularly in dynamic or sensitive areas



MEAMO LABS SILHOUETTE HA BODY — **RECOMMENDED APPROACH:**

- Volume of added solution: from 10 ml and more, depending on the anatomical area, injection technique, and desired effect
- For areas requiring firmer fixation
 (e.g., upper buttocks), minimal dilution is used
- In areas where maximum flexibility and natural feel are important (e.g., thighs, sides, outer edges of the buttocks), greater dilution may be applied

After adding NaCl, the product **should be thoroughly mixed** to achieve a smooth, lump-free texture.

Tip: To facilitate injections and ensurecomfortable distribution, you can slightly warm the syringe with the diluted product to body temperature. A warm filler passes more easily through the cannula and integrates more smoothly into the tissue



MEAMO LABS SILHOUETTE HA BODY — **RECOMMENDATIONS FOR VOLUME AND INJECTION FREQUENCY:**

- The duration of the result is individual and depends on lifestyle, metabolism, and personal characteristics. In patients with a high level of physical activity and low subcutaneous fat, the effect may last for a shorter time than in others. On average, to maintain the achieved volume, a repeat procedure may be required every 3-6 months
- The interval between procedures
 is determined by your doctor or injection specialist
- The specific injection schedule is tailored individually, depending on the area that requires correction and the severity of the issue



What to Do After Injection Sessions?

Minimal Recovery Period!

Immediately after the application of Meamo labs Silhouette HA Body, the following effects may be observed in the injection area:

Swelling



Numbness or redness



Small bruises



Hardened areas



Post-procedure recommendations

It is also recommended to follow these guidelines:

Avoid



touching or applying cosmetics to the treated area for 24 hours

Do not use



scrubs or peels on the treated skin for one week

Avoid



intense physical activities, visiting pools, saunas, and tanning beds **for 2 days**

If you experience discomfort



you can apply ice to the injection areas to speed up recovery and reduce discomfort

How Quickly Can You See the Results of Meamo labs Silhouette HA Body Injections?

The Meamo labs Silhouette HA Body injection procedure takes only about 30 minutes. The effect of volume restoration is immediate. However, the results will be most noticeable after 1-2 days when the side effects from the injections subside.

Additionally, fibroblast stimulation and new collagen synthesis will provide a cumulative effect. This means that results will continue to improve over time, and the lifting effect, as well as enhanced skin firmness and elasticity, will gradually become more pronounced.

How to Store Meamo labs Silhouette HA Body?

Precautions for the safe storage and use of the next-generation injectable filler Meamo Labs Silhouette HA Body

Storage of an opened vial is not allowed.

Once opened, the sterility of the product is compromised, making further use unsafe.



The product is intended for single-use within one procedure.

Any leftover filler after opening should be discarded.



- Important: Reusing an opened product, even with proper temperature control and aseptic conditions, may lead to infection, inflammatory reactions, or other complications.
- Recommendation: Always use the entire volume of the product immediately after opening. This ensures maximum effectiveness and safety for the patient.







